

Instructions for packaging and identification of samples

All samples must be packed using the Triple layer system

- 1. Leakproof primary receptacle wrapped in sufficient absorbent material to absorb the entire contents**
- 2. Leakproof secondary receptacle (Ziplock Bag)**
- 3. Outer packaging (rigid)**

Enclose paperwork between secondary and outer layer

Either the primary or secondary must be able to withstand 95kPa pressure differential

Labelling:

Packaging should have the following clearly marked on it:

- **Delivery address**
- **Senders details**
- **Emergency contact details**
 - **name and telephone number**
- **Samples should be labelled with the following:**
 - **“Exempt animal specimens”**
 - **And the following UN Biological specimens mark:**



Farmlab Diagnostics, Elphin, Co. Roscommon

Tel: + 353 71 9630792 ~ Fax: + 353 71 96 30787 ~ email: info@farmlab.ie ~ www.farmlab.ie