

Instructions for packaging and identification of samples:

All samples must be packed using the Triple layer system

1. Leakproof primary receptacle wrapped in sufficient absorbent material to absorb the entire contents
2. Leakproof secondary receptacle (Ziplock Bag)
3. Outer packaging (rigid)

Enclose paperwork (**F#514-1 FarmLab Diagnostics Sample Submission Form**) between secondary and outer layer.

Either the primary or secondary must be able to withstand 95kPa pressure differential.

Labelling:

Packaging should have the following clearly marked on it:

- Delivery address
- Senders' details
- Emergency contact details
 - name and telephone number
- Samples should be labelled with the following:
 - "Exempt animal specimens"
 - And the following UN Biological specimens mark:

